LEDRAGOMMA®
ITALIAN QUALITY SINCE 1950

Ledragomma Srl
via G. De Simon snc
33010 OSOPPO / UD / Italy

T. +39 0432/986049
F. +39 0432/986255
info@ledragomma.com
info@ledragommapec.com

Capitale Sociale Euro 10.000,00 i.v.
R.E.A. UD - 277758
C.F./P.IVA - TVA-VAT: IT 02653310306

IT

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BRAND OF
PROPERTY

WWW.TONKEY.IT

ISO 9001 Quality System
ISO 14001 Environmental Management System
ISO 13485 Medical Devices

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READ CAREFULLY BEFORE EACH USE OF THE BALL

DIRECTIONS AND WARNINGS

This document is the **essential guide** for a proper and safe usage of the **PendyBall by Ledragomma Original Pezzi®**: it contains the **necessary information** in order to use the product with **full satisfaction** and in the respect of the **safety rules** during the life of the product itself and/or its parts.

PROVIDED USAGES - LIMITS AND EXCLUSIONS

1. This ball has been designed and arranged to be used in various fields, such as sport, fitness, gymnastics but also in the medical. It must not be used for different purposes than the ones stated by **Ledragomma**. It must be handled with the utmost care and attention.
2. With the exception of the prohibition contained in section 4, the weight in excess of that recommended will impede the performance of the PendyBall by Ledragomma original "pezzi"® : Ø 21.5" up to 154 lbs., Ø 25.5" up to 198 lbs. and Ø 29.5" up to 231 lbs.
3. **IT IS NOT A TOY** and, in any case, it's not suitable for children under 36 months : small parts included. Choking hazard.
4. **DO NOT perform exercises with barbells, dumbbells or any kind of weights on the ball.**
5. Do not try to remove the large cap attached to the pendulum.
6. Although the pendulum inside the ball is made of very strong material, it must be handled with the utmost care and attention and used only for those purposes indicated by Ledragomma. **When NOT USED**, the ball can be placed on the **Stopperball®** (provided with the packaging), with the pendulum in vertical position, as in use. Using the PendyBall by Ledragomma original "pezzi"® for purposes that do not comply with the provided instructions, may cause damage to the pendulum and its structure.

PACKAGE CONTENT

7. The package contains a ball, a DVD, a Stopperball®, a yellow valve and two white plugs.

INFLATION

8. To inflate the ball, air mattresses pumps, bicycle pumps (only in this case use the yellow valve included in the package) or air compressors must be used. **DO NOT inflate by mouth**.

FIRST USAGE: MODE AND PROCEDURE TO FOLLOW

9. By first usage, proceed in accordance to the following directions:
 - 9.1) start to slowly inflate the ball from the small hole (located on the ball's lower pole), pausing occasionally. To inflate the ball, use the tools specified in section 8,
 - 9.2) don't exceed the 90% of the maximum diameter, indicated on the ball itself and on the packaging,
 - 9.3) close the hole with one of the two provided white plugs (the second plug is a spare),
 - 9.4) wait 24 hours before inflating it to its maximum diameter and start to use it.

DIRECTIONS AND WARNINGS FOR THE FOLLOWING INFLATIONS

10. For the following inflations, never exceed the prescribed and indicated maximum diameter. If an air compressor is used, proceed slowly while approaching the prescribed maximum diameter. The plug can be removed using a coffee spoon or the Ledragomma Plug remover.

PRECAUTIONS TO ADOPT BEFORE EACH USAGE

11. Before each use, check the surface of the ball and don't use it if it shows cuts, abrasions, holes or any other kind of damage. In all these cases, the safety of the ball is compromised and the ball itself could instantly tear, causing the user possible injuries.
12. Before each use of the ball, wash it with water and delicate toilet soap and remove the more obstinate stains with diluted alcohol.
13. The rotational movement of the pendulum may cause small amounts of air to leak from the large cap. Therefore, before each use, check the diameter and, if necessary, slightly inflate the ball.

USAGE TEMPERATURE

14. Caution: the ball must be kept at room temperature (18-20°C). In case of particularly low temperature, proceed with inflation only after letting the ball reach the above mentioned room temperature.

USED SURFACES

15. The ball must be used on soft surfaces, such as carpets, rugs, exercise mats. Never use the ball in presence of sharp objects or close to heating elements; when not in use, never leave the ball directly exposed to the sun.

REPAIR WARNINGS

16. If the ball shows holes or cuts, don't try to repair it: self made repairs invalidate the integrity of the product causing serious risks for the safety and health of the user.

TRACEABILITY

17. A traceability code of 4 digits is marked on the ball; it indicates the month and year of production for this item. Please keep this code in the case of a claim, in order to identify the production time and the batch materials.

LEDRAGOMMA®
ITALIAN QUALITY SINCE 1950

Ledragomma Srl
via G. De Simon snc
33010 OSOPPO / UD / Italy

T. +39 0432/986049
F. +39 0432/986255
info@ledragomma.com
info@ledragommapec.com

Capitale Sociale Euro 10.000,00 i.v.
R.E.A. UD - 277758
C.F./P.IVA - TVA-VAT: IT 02653310306

DISCLAIMER

18. Ledragomma Srl declines any liability for damages to persons, animals or objects or to the environment arising from the a non consistent use or maintenance to the precautions and directions indicated in this leaflet accompanying the product.

ATTENTION

In case of serious accident, please, inform the manufacturer and the local competent authority.

DISPOSAL

19. The device will have to be disposed of according to the law currently in force.

NOTE

20. Colours and decorations may vary according to manufacturer.

Made in Italy by:

Ledragomma Srl - Zona Industriale – Via G. De Simon - 33010 Osoppo (Ud) – Italy
Tel.+39 0432/986049 – Fax +39 0432/986255 - info@ledragomma.com - info@ledragommapec.com - www.tonkey.it

SYMBOL LEGEND

	Manufacturer
	Keep away from light and heat sources
	CE marked Medical Device MDR 745/2017
	Pay attention
	Lot number

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